
**CERTIFIED**  
(EU) 2016/425  
EN 149:2001  

—CE—

## **FFP NR – Non reusable**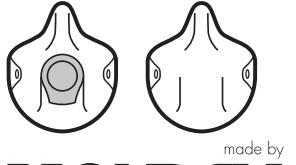

# MOLDEX

R

**GB - Instructions for use** Use Protects from solid and liquid aerosols to EN 149:2001+A1:2009 NR: for single use, dispose at end of shift. **D:** Dolomite clogging test passed. **Questies:** Contact your safety officer or Moldex. To select the right respirator and protection level observe national regulations. Check before use/fitting See pictograms on box. Use mask only if not damaged. Perform a fit check! Cover the mask with both hands and exhale sharply. If leakage is detected, check position of mask – if necessary shorten strap. Only enter workplace when proper fit is achieved. Note: Facial hair can prevent adequate fit. **Warning** The wearer must be instructed in the proper use of the mask. No metal parts that can cause sparks in explosive atmosphere. Does not protect the wearer against gases / vapours, or atmospheres with less than 19.5% oxygen. Do not enter unventilated narrow rooms and shafts with filtering face pieces. Do not use against contaminants / concentrations unknown or immediately dangerous to health. Leave area immediately when breathing becomes difficult or other distress occurs. Dispose respirator when damaged or breathing resistance increases strongly. Never modify or tamper with mask. Incorrect use or not following the user instruction can reduce

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Moldex/Metric AG & Co. K